BIOPSY PUNCH

EN Instructions for Use

1. Device Description

The device consists of blade and handle.

The intended purpose of this sterile disposable cutaneous biopsy punch is to perforate skin tissue for the examination, diagnosis and treatment of skin diseases.

2. Instructions for Use

For use by healthcare professionals only.

Push and rotate the blade vertically against the subject tissues to perforate or collect them.

For the device with plunger, push the plunger to extrude the remaining tissues inside the blade.

Manipulate with attention to the thickness of the skin tissue and other immediate tissue.

After use, dispose of the device properly as medical waste to prevent infection.

3. Remarks

Do not use on patients who present sensitisation or allergy.

Reuse of this device presents a risk of infection to the patient.

Handle with care so as not to injure yourself by touching the edge of the blade.

If the cutting edge comes in contact with a foreign object, replace it with a new one.

Do not use the device if any damage or other abnormalities are observed.

As this device contains Co (cobalt), sustained contact may result in adverse effects; use only for the intended purpose.

Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority.

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KAI INDUSTRIES CO., LTD.
KAI EUROPE GmbH