DERMAL CURETTE

EN Instructions for use

1. Device Description

The device consists of blade and handle.

The intended purpose of this sterile disposable cutaneous biopsy curette is to curette skin tissue for the examination, diagnosis and treatment of skin disease.

2. Instructions for Use

For use by healthcare professionals only.

Push the blade against the subject tissues to curette and collect them.

After use, dispose of the device properly as medical waste to prevent infection.

3. Remarks

Do not use on patients who present sensitisation or allergy.

Reuse of this device presents a risk of infection to the patient.

Handle with care so as not to injure yourself by touching the edge of the blade.

If the cutting edge comes in contact with a foreign object, replace it with a new one.

Do not use the device if any damage or other abnormalities are observed.

As this device contains Co (cobalt), sustained contact may result in adverse effects; use only for the intended purpose.

Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority.

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KAI INDUSTRIES CO., LTD.
KAI EUROPE GmbH